

## Toilet Seat Elevator (elongated)

*These instructions are guidelines only. Use only as instructed by your healthcare provider.*

### What is a toilet seat elevator used for?

A toilet seat elevator (**Figure 1**) raises the height of the standard elongated toilet seat in your home to assist those with bending or sitting difficulties. Easy to install, using only a screwdriver, the elevator is fitted under your own toilet seat.



**Figure 1**

### How Do I Install a Toilet Seat Elevator?

- **\*Step 1:** Remove the existing toilet seat by unscrewing the two bolts and nuts. For most toilets you must reach behind and underneath the toilet bowl to unscrew the nut while keeping the bolt on the attachment portion of the seat stationary. (Note; bolts on top may be covered with a cap which will need to be removed first).
- **Step 2:** Place the toilet seat elevator on top of the toilet such that the two slots on the back align with the holes of the toilet.
- **Step 3:** Place the toilet seat back on top of the seat elevator such that the two holes of the toilet seat align with the two slots of the seat elevator.
- **Step 4:** Pass the two long bolts provided through the holes of the toilet seat, seat elevator and the toilet. Tighten both bolts underneath the toilet bowl with the washers and nuts provided.  
\*See diagrams over page.

**! WARNING:** Ensure that the seat is correctly and securely fastened in place before using.

**! WARNING:** If recommended by your health care professional the user should be supervised or assisted when using this product.

**! WARNING:** Stability of the seat and tightness of bolts should be checked before use.

### How Do I Care for the Toilet Seat Elevator?

- The toilet seat elevator can be cleaned with detergent and warm water, rinsed and then wiped dry. Do not use abrasive powders or scouring pads.
- Do not use solvents, such as acetone, lacquer thinner or turpentine.



**Installation instructions:**


**Step 1:**



**Step 2 & 3:**



**Step 4:**

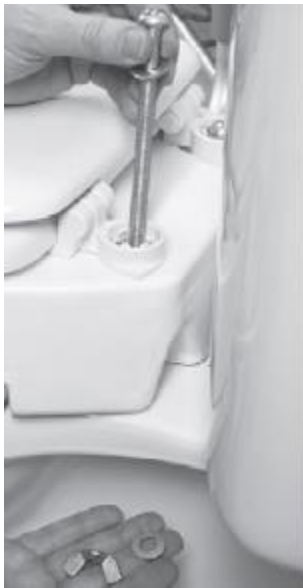

*Please ensure that equipment returned to the Red Cross is clean and in good condition.*

